



VentLab Disposable Bacterial/Viral Filter (HEPA Filter) Ventilation Circuit - Guidance

Product: VentLab disposable bacterial / viral Filter: (bi-directional valve)

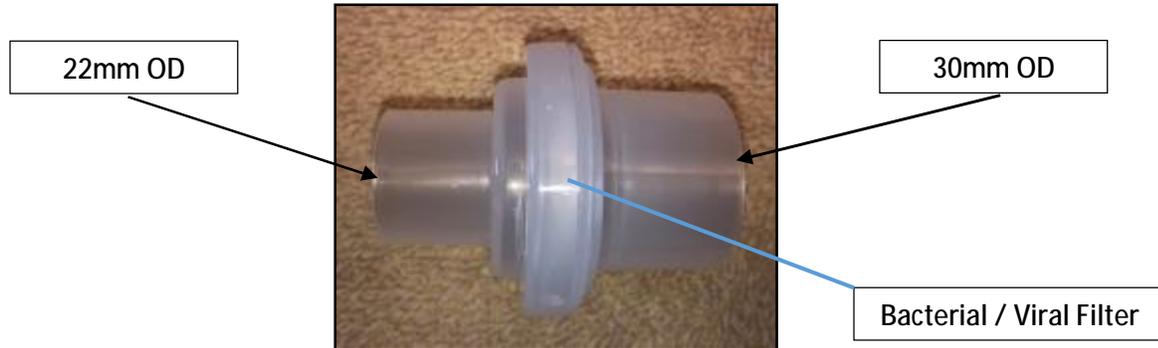

- A. HEPA Filter Attachment to Hand-held Nebulizer:
(Patient mouthpiece > Nebulizer T-piece > HEPA Filter)

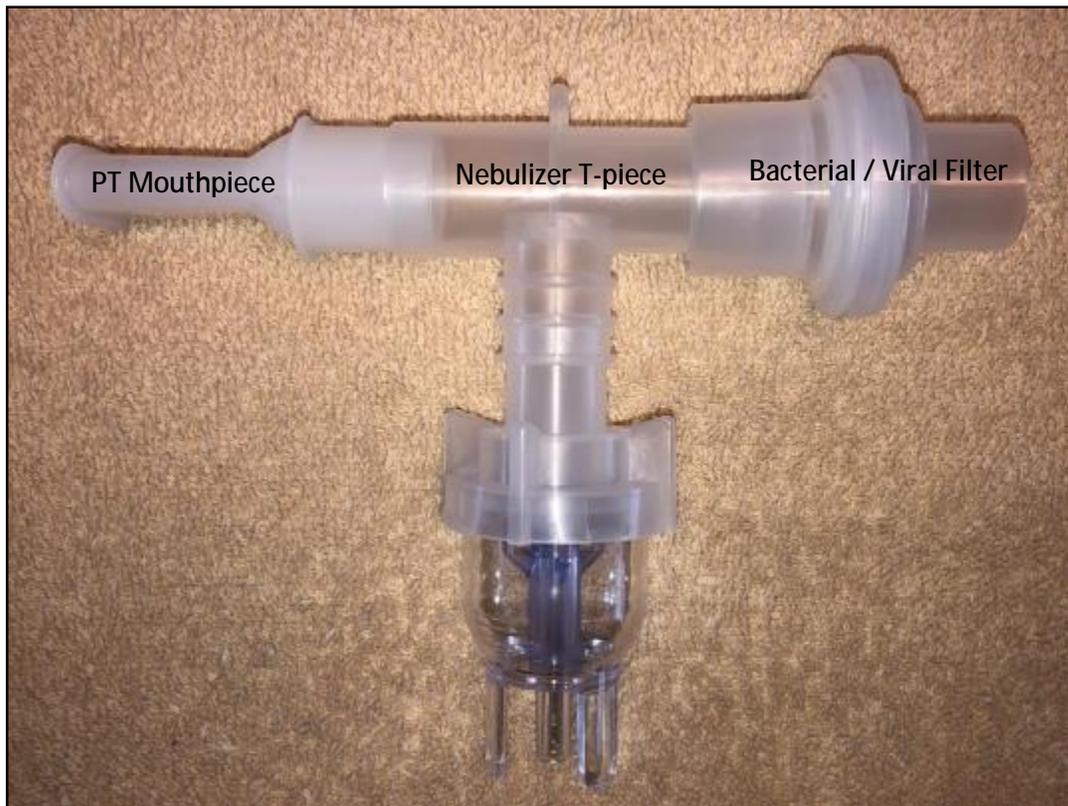

VentLab Disposable Bacterial/Viral Filter (HEPA Filter) Ventilation Circuit - Guidance

B. Bag-Valve-Mask/Tube for ventilation with HEPA Filter attachment:

Without ResQPOD: (Mask / Tube > HEPA Filter > EtCO₂ Filterline > BV Device)

With ResQOD: (Mask / Tube > ResQPOD > HEPA Filter > EtCO₂ Filterline > BV Device)



C. CPAP with HEPA Filter Attachment:

1. Without Nebulized Therapy: (Mask > HEPA Filter > CPAP circuit)

2. With Nebulized Therapy: (Mask > Nebulizer set-up w/T-piece > HEPA Filter > CPAP circuit) NOTE: The in-line nebulizer attachment within the CPAP circuit will NOT be used***

